

■ Company introduction

1. Overview

- Company name : TLVision Co., Ltd.
- CEO name : Youn Jun Chae
- Capital : USD 200,000
- Main product : Touch panel , Health mask
- Home page : www.tlvision.co.kr
- E-mail address : dcyang@tlvision.co.kr
- Head office and R&D : Anseong city at Korea



■ Company introduction

2. History

- 2015.01 : Company establishment
- 2015.12 : Registration of 2 patents
ISO 9001 & ISO 14001 certificate
- 2016.02 : Establishment of oversea factory in China(Weihai)
- 2016.03 : Development of MP3 player touch panel
- 2017.01 : Establishment of R&D center
- 2018.01 : CE certificate
- 2019.10 : Development of 86inch touch panel
- 2019.11 : Completion of new factory
- 2020.02 : Starting new business for health mask




Company introduction



3. View




4. Location

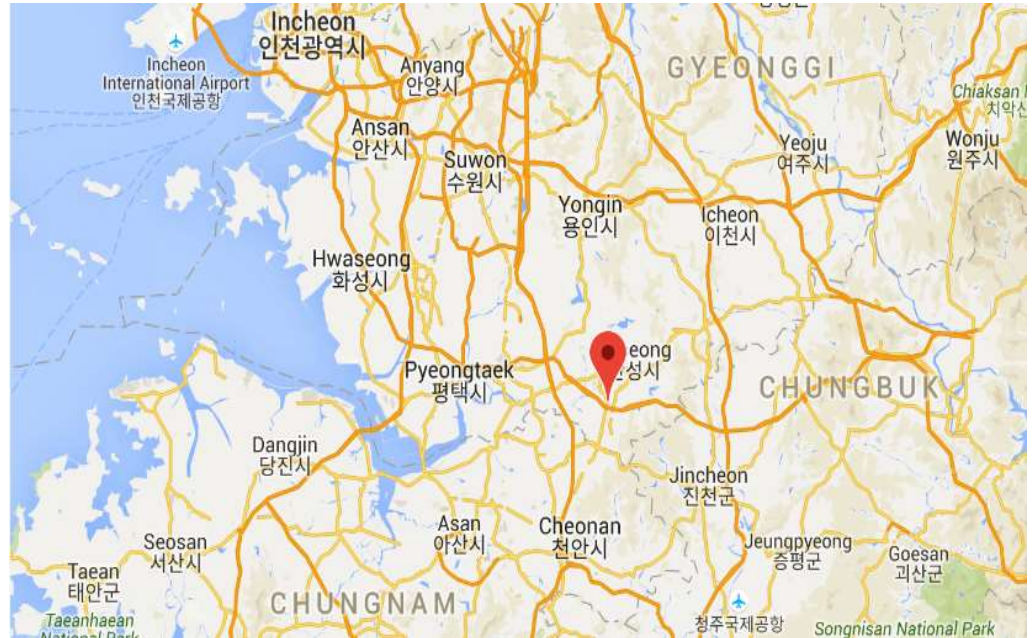

■ Product description

- Product : Disposable health mask
- Model and Brand : KF94 & BT Smith
- Image :

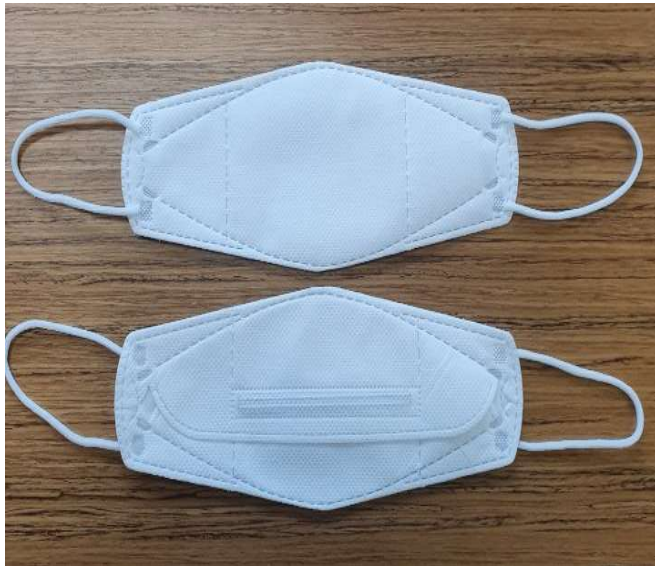

■ Product description

Feature

- Use non-woven fabric in a 4-layer
- 95% blocking fine dust and viruses
- Certified by Korea FDA
- More confident in quality than low priced Chinese
- No ear pain using a flexible ear-band.
- 36 months guarantee from production date

Application

- Fine dust and Viruses



■ How to use



The nose clip part is facing up, the side without nose clip part is facing down, and the ear-band are pulled on each hand.



Wear the mask against the chin at first, and pull the ear-band back to the ear and hand it on the ear until it feels comfortable.



Place your fingers in the middle of the nose clip and then press to conform the shape of the nose bridge.



Put your hands on your face to check the tightness of the mask and adjust it again if need.



■ Precautions

- Do not place the mask on the chin but cover both the nose and mouth.
- Do not use the mask more than once and scrap the used mask immediately.
- It is better not to touch the surface of a mask that may be contaminated.
- Touch only the edges, straps, or bands of the mask when taking off the mask.
- Do not crumple the mask when wearing it and storing it.
- Do not take off the mask indoors and continue to wear it

